
- 1. Pull nylon over cast
- 2. Nail molding dummies onto cast. Please note: the dorsi-assist dummies need to be placed as shown

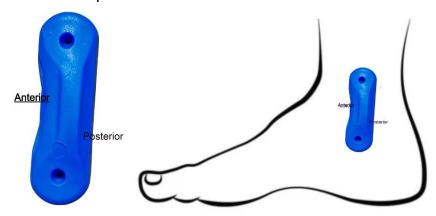

- 3. Pull plastic over cast
- 4. Mark trim lines and cut the AFO along the trim lines
- 5. Remove AFO then dummies (dummies can be reused if they are still in good shape)
- 6. Place joints into the joint's cavities on AFO
- 7. *Use a hot rod to go through the threaded holes on joints and then poke through the AFO
- 8. Then install screws provided to secure the joints, use medium strength Loctite.
- 9. Apply the friction reduction stickers to the 4 threaded holes
- 10. Use the provided bumper as a range of motion limiter.